



# Fraxel re:store<sup>®</sup> **DUAL** system

Treatment using the 1927 nm wavelength

PATIENT DIARY



## CASE STUDY

### Patient

A 47-year-old female Executive Assistant seeking treatment for sun spots, which have worsened over the years. She has an active outdoor life style and enjoys hiking and gardening. Her dermatological history is positive for rosacea, for which she uses Metrogel® as needed.

### Physician

Steve Struck, M.D. board-certified plastic surgeon practicing in Atherton, California.


### Preparation

A topical anesthetic ointment was applied one hour prior to procedure and removed immediately before the treatment.

### Follow-up care

Gentle cleansing (Cetaphil® gentle facial cleanser), Moisturizing (Aquaphor® healing ointment) and Sun protection (Neutrogena® SPF® 70®).

BEFORE

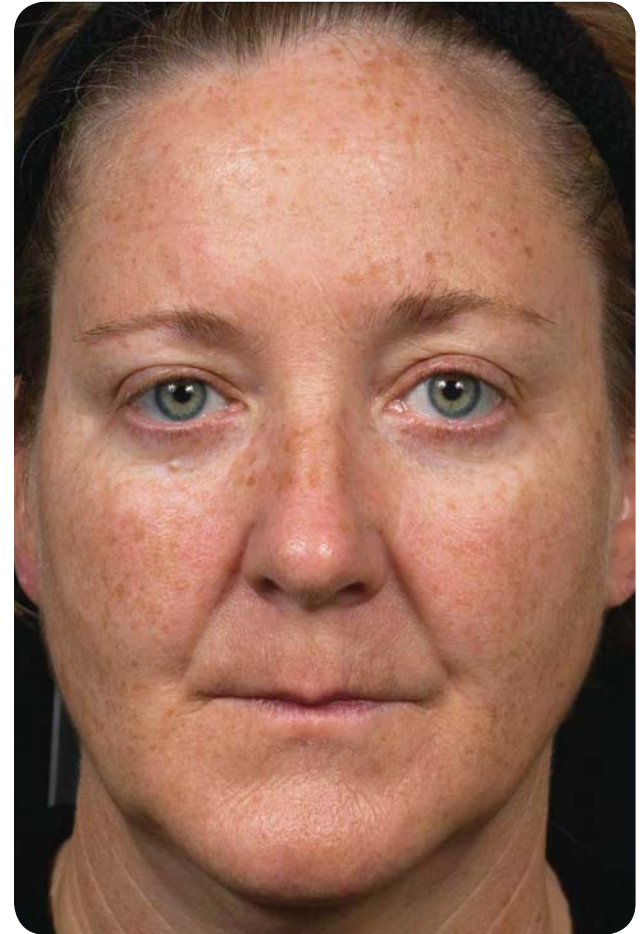

## TREATMENT EXPERIENCE

### *Patient*

The numbing gel really works, I'm totally numb. The procedure was completely painless and actually felt nice with the cooling air that sort of felt like cool mist. My face started to tingle about 10 minutes after the procedure. When the numbing gel wore off my face felt hot and very tingly, like a "fresh" sunburn, this lasted about an hour. I took Ibuprofen to ease the sunburn feel.

### *Physician*

Patient was relaxed and tolerated the procedure very well. She reported a pain score of 1 on the 0 to 10 scale, 10 being the worst pain. Mild erythema (redness) and almost no edema (swelling) immediately post procedure, as normally expected.

## IMMEDIATELY POST TREATMENT



## DAY 1

### *Patient*

*I slept well (last night), not uncomfortable at all. A bit itchy, I can't get enough moisturizer on my face!*

### *Physician*

*Pruritus (itching) is a common side effect, which can be relieved by the use of an ice pack and liberal application of moisturizer. It is not uncommon for the patient to look a bit more erythematous (red) than they did immediately after treatment.*



## DAY 2

### *Patient*

*I am pretty brown, skin is very dry, has a sandpaper-like feel. At 9:00 pm, I saw first signs of peeling.*

### *Physician*

*If you look carefully, you can see the individual laser "dots" on the skin surface. This contributed to the overall darkened appearance.*



### DAY 3

**Patient**

*Wow, I am officially peeling!  
And it feels so much better.*

**Physician**

*Desquamation (peeling) usually  
occurs day 3 – 4 after a face  
treatment. At this time, patient  
should refrain from picking or  
manually exfoliating  
the treated area to allow  
natural healing.*



### DAY 4

**Patient**

*It's almost all off, just a bit still  
along the jawline, hairline and  
nose tip. All the pigmentation  
has faded away.*

**Physician**

*Desquamation tends to  
start from center face,  
as observed here.*



## DAY 5

### *Patient*

*Everything is off, skin looks luminous, baby soft. People who have not seen me in a while, noticed a "glow" about me.*

### *Physician*

*Patients, in general, feel much better when desquamation (peeling) is complete.*



## 1 WEEK

### *Patient*

*I am so glad that I did this and it's a pretty amazing result for just 7 days later. Now it's smooth, clear, slightly pink with a healthy look. I will be diligent with the SPF and hat wearing when I exercise or work in the yard.*

### *Physician*

*Patient still has some redness but nothing significant. Sun avoidance/sun protection is essential after a laser resurfacing procedure, both to protect the skin and prolong the results.*



## WEEK 2

### *Patient*

*I have not put any make up on my face at all. Just moisturizer and it's a great feeling. My skin is still very smooth and my cheeks have a healthy pink glow.*

### *Physician*

*All redness is resolved at this time. Patient may return to her normal skin care routine.*




## 1 MONTH

### *Patient*

*I have one small area of pigmentation that is much lighter than before but remains; I have been told it is due to the depth of the pigmentation. Overall I am delighted with the outcome and have recommended the procedure to several friends and family!*

### *Physician*

*At this time, a series of 1550 nm treatments may be considered to further improve some of the deeper pigmentation, as well as long term collagen remodeling to address deeper wrinkles. In addition, a good skin care regimen and sun protection will help to maintain clearance of superficial pigmentation from the 1927 nm treatment. Individual results may vary.*

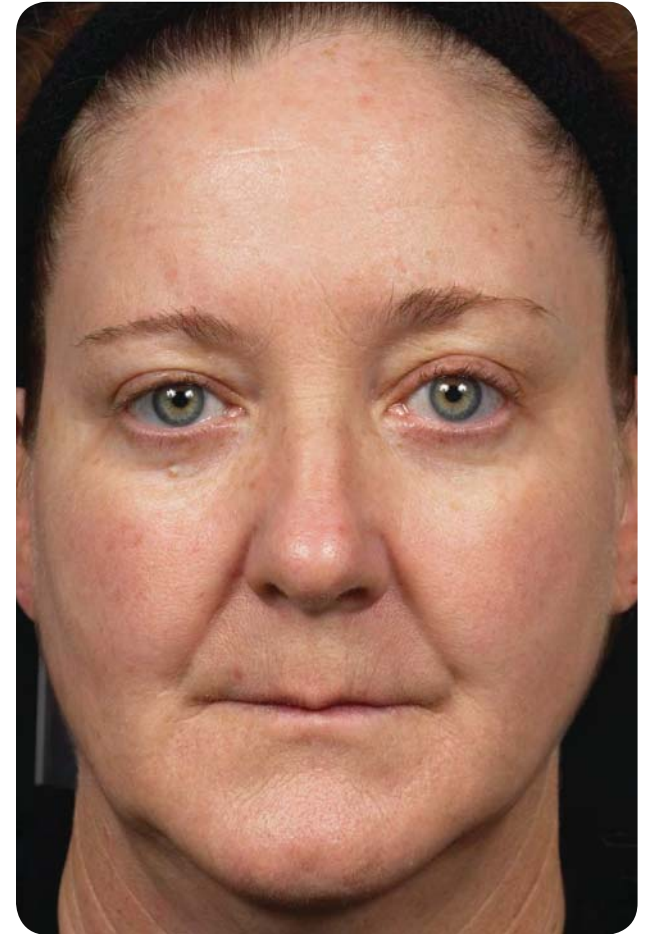

BEFORE



1 MONTH AFTER







1927 nm is indicated for use for soft tissue coagulation.

MK2092B ©2010 Solta Medical, Inc. All rights reserved. Fraxel, the Fraxel logo and Fraxel re:store are trademarks or registered trademarks of Solta Medical, Inc. or its subsidiaries. Other names and brands may be claimed as the property of others.